

Analytical and Clinical Validation of a Methylation-Based cfDNA Assay for Early Detection of High-Mortality Cancers Using a Cost-Efficient Reflex Approach

Supawadee Chawanthayatham, PhD¹; Joshua Hubbell¹; Nate Ruffle-Deignan¹; Kristy Schlueter-Kuck, PhD¹; A. Gregory DiRienzo, PhD¹; Johanna B. Withers, PhD¹; Dorna Kashef, PhD¹; James Sun, PhD¹; Jocelyn Charlton, PhD¹; Rosemary T. Witkowski, PhD¹; Daniel Sisco¹; Kieran I. Chacko, PhD¹; Hutan Ashrafiyan, MD, PhD¹; Feras Hantash, PhD¹; Avni Santani, PhD¹*
¹Harbinger Health, Cambridge, MA * [Corresponding author: asantani@harbinger-health.com](mailto:asantani@harbinger-health.com)

INTRODUCTION

- Population screening for early cancer detection has the potential to improve clinical outcomes.
- HarbingerHx (HHx) is a blood-based cell free DNA (cfDNA) methylation assay for multi-cancer early detection (MCED), evaluating CpG methylation status of cfDNA fragments across a broad range of loci to detect cancer-associated signals and tissue localization.
- The HHx two-tier reflex design uses a cost-effective initial screen (Tier I) to rule out cancer-negative samples, with non-negative samples advanced to a larger Tier II panel for confirmatory analysis, providing high specificity and tissue-of-origin (TOO) prediction.

OBJECTIVES

- To validate the HHx assay for population screening of asymptomatic individuals with high-mortality cancers (lung, upper gastrointestinal, head and neck, hepatobiliary, and pancreatobiliary) using Tier I and Tier II workflows, evaluated independently and in combination within a multi-tier reflex framework.
- To assess analytical and clinical performance, including sensitivity, specificity, precision, limit of detection (LOD), preanalytical interference, cfDNA stability, and tissue-of-origin (TOO) prediction accuracy.

METHODS & MATERIALS

- We performed a case-control study (NCT05435066) collecting whole blood from treatment-naïve cancer patients and individuals without cancer aged 45-80, from which cfDNA was subsequently extracted for analysis.
- cfDNA inputs (10-30 ng) were subjected to bisulfite conversion followed by library construction. Indexed libraries were enriched for cancer-specific methylation biomarkers using Tier I (156 kb) and Tier II (18.6 Mb) probe panels and subsequently sequenced.
- Machine learning (ML) models, trained on a separate >5,000 mutually exclusive sample (independent from the validation cohort) generated ML-derived output scores, binary classifications, and tissue-of-origin (TOO) assignments.

Study	Samples and Method Description
Clinical Sensitivity	466 cancer; determine the proportion of patients correctly classified as cancer. Evaluated for Tier I, Tier II and reflex workflows.
Clinical Specificity	1,067 non-cancer; determine the proportion of patients correctly classified as non-cancer. Evaluated for Tier I, Tier II and reflex workflows.
Tissue of origin accuracy	Evaluated using cancers with a cancer positive result based on the Tier II or reflex result.
Analytical Sensitivity (LOD)	22 cancer; A series of tumor content (TC) of 0.05-0.50% prepared by diluting individual cancer cfDNA into pooled non-cancer cfDNA.
Analytical Precision	17 cancer and 10 non-cancer, each sample in triplicate per run, inter-assay assessed across 4 runs, inter-operator/instrument/day were evaluated across at least 2 runs.
cfDNA Stability	25 cancer tested across freeze-thaw and long-term storage time points; assess the QC metrics and assay performance.
Interference	gDNA spiked into cfDNA (20-80%), Hemoglobin spiked into healthy plasma (100-1,000 mg/dL); assess the QC metrics and assay performance.

Table 1. Overview of Analytical Validation Studies.

REFERENCES

- A. Gregory DiRienzo, Ele Massaad, Hutan Ashrafiyan. Tissue-specific predictive performance: A unified estimation and inference framework for multi category screening test, arxiv.org/abs/2505.21482. May 2025
- JP Gregg, F Michor, et al. (2023) Novel blood-based assay demonstrates high sensitivity for early stage multi-cancer detection. Journal of Clinical Oncology, Volume 41, Number 16 suppl.

RESULTS

Clinical Performance and TOO Accuracy

- The Tier I panel, designed as a cost-efficient initial screen, achieved 82.8% specificity while maintaining strong overall sensitivity at 80.3%, effectively ruling out non-cancer samples early in the workflow.
- The Tier II panel reached 98.8% specificity, establishing a robust benchmark for confirmatory testing by minimizing false positives and unnecessary follow-up.
- The reflex workflow, which integrates both tiers sequentially, provides balance of sensitivity, specificity, and cost efficiency. By advancing only non-negative Tier I samples to Tier II, the approach significantly reduces sequencing demand and overall testing cost while preserving high analytical and clinical performance, making it well suited for population-scale applications.
- Tier I demonstrated sensitivity ranging from 76 to 90%, while Tier II showed sensitivity between 65 and 90% across different high-mortality cancer subgroups. The Reflex test achieved sensitivities of 54% to 85%, demonstrating consistent and accurate detection across cancer readout subgroups.

Metric	Tier I	Tier II	Reflex
Specificity	82.8% (±95% CI;80.4%-84.9%) (n=879/1062)	98.8%(±95% CI;97.9%-99.3%) (n=1037/1050)	98.8%(±95% CI;98.0%-99.3%) (1036/1049)
Sensitivity	80.3% (±95% CI;76.4%-83.6%) (n=374/466)	69.8%(±95% CI;65.5%-73.8%) (n=324/465)	68.8%(±95% CI;57.5%-69.7%) (n=319/464)
TOO Accuracy	N/A	84.5%(±95% CI;80.2%-88.1%) (n=274/324)	83.7%(±95% CI;79.6%-87.8%) (n=267/319)

Table 2. Analytical Specificity, Clinical Performance, and TOO Accuracy (±95% CI).

Analytical Sensitivity: LOD

Based on a sample titration series, the assay achieved an empirical LOD₉₅ of

- 0.055% TC for Tier I at 80% specificity
- 0.060% TC for Tier II at 98.5% specificity

Demonstrating high analytical sensitivity across both tiers and the ability to detect low TC with robust performance.

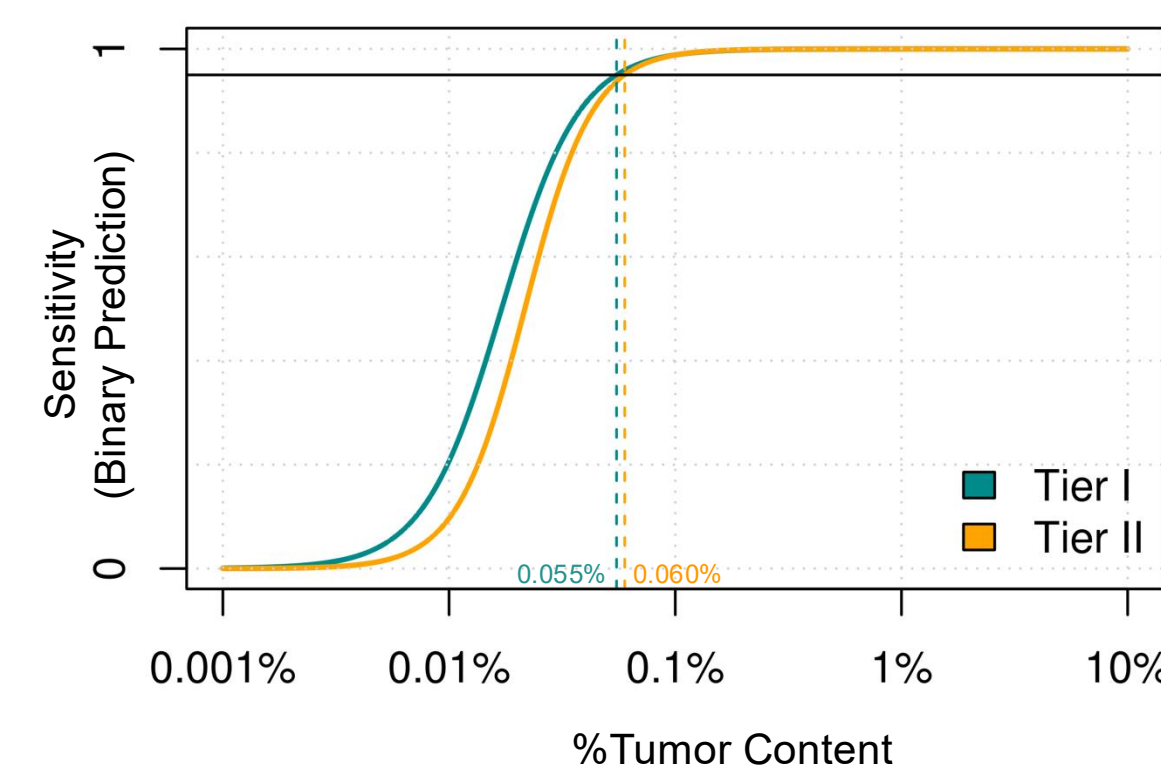


Figure 1. The Empirical LOD₉₅

Analytical Precision

- The HHx assay demonstrated strong reproducibility across all precision types.
- Tier I showed >96% and Tier II achieved >97% within-tier replicate concordance, based on consistent classification results across technical replicates.
- Confirming highly consistent performance across runs, operators, instruments, and days.

Precision Type	% Concordance of Classification	
	Tier I	Tier II
Within-Run	83/86 (96.50%)	81/83 (97.50%)
Between-Run	248/254 (97.6%)	320/324 (98.80%)
Inter-Operator	153/156 (98.1%)	161/162 (99.4%)
Inter-Instrument	153/156 (98.1%)	160/162 (98.8%)
Inter-Day	153/156 (98.1%)	160/162 (98.8%)

Table 4. Summary of Precision Metrics.

cfDNA stability

96% (24/25) of cfDNA samples showed classification concordance when processed twice after long-term storage (5 months to 1 year at -80°C), and multiple freeze-thaw cycles. Methylation patterns were highly consistent, confirming cfDNA stability and HHx workflow robustness under routine handling.

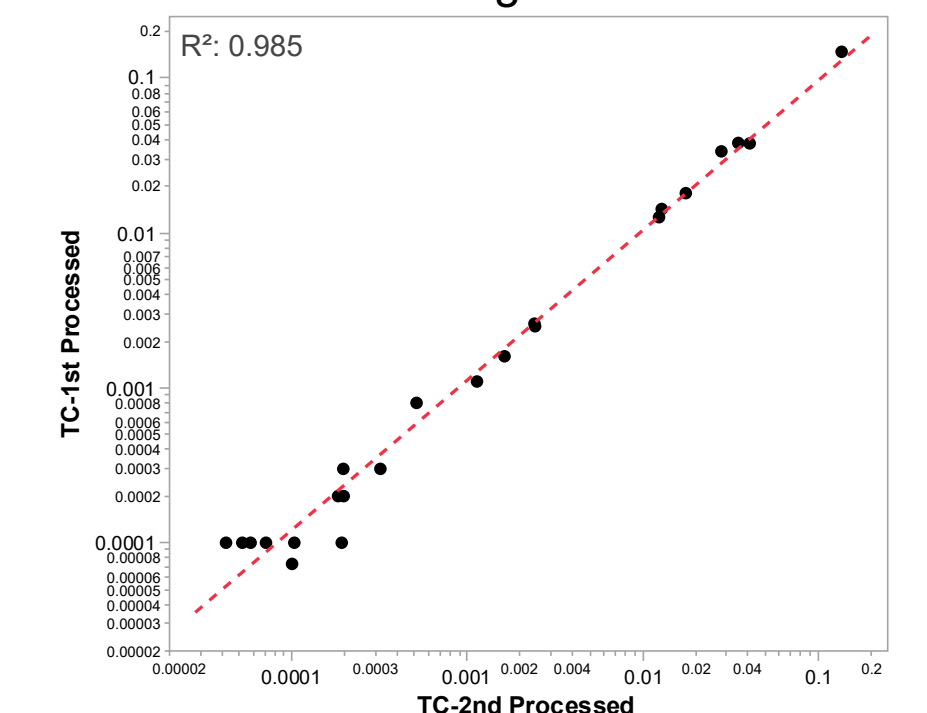


Figure 2. Methylation signal correlation (R² = 0.985) between samples processed at initial and repeat timepoints after long-term storage.

Interference

Assay performance remained stable with hemoglobin levels up to 1000 mg/dL and gDNA contamination ≤ 40%. Methylation patterns and QC parameters were consistent across all replicates and tiers, achieving 100% classification concordance. These results demonstrate that, under the tested conditions, the assay maintained robust performance and reliability against common pre-analytical interferences.

CONCLUSIONS

- The HHx cfDNA methylation assay demonstrated robust analytical and clinical performance across both Tier I and Tier II workflows. The two-tier reflex design effectively combines a cost-efficient initial screen with a highly specific confirmatory test that includes tissue localization, achieving exceptional specificity suitable for population-level testing.
- High reproducibility, low LOD₉₅, and resilience to preanalytical variation confirm the assay's reliability under routine laboratory conditions. Collectively, these results establish a scalable and precise approach for MCED, reinforcing the clinical utility of the HHx assay for population-based testing and improving early diagnosis of high-mortality cancers.

DISCLOSURES

This study was sponsored by Harbinger Health, Cambridge, MA

ACKNOWLEDGEMENTS

We gratefully acknowledge all participants for their contributions, without whom this research would not have been possible. We also thank all cross-functional teams for their dedicated efforts that made this work and the resulting data possible.